



K2 EDTA TUBES

PROVETTE CON K2 EDTA

STERILE R

CODE codice	SIZE dimensioni	DESCRIPTION descrizione	DRAWING aspirazione	COLOUR colore		SHELF-LIFE scadenza	PACKAGING confezion.
13501	13 x 75 mm	K2 EDTA K2 EDTA	1 ml	Lavender Viola		18 months mesi	100 / 1000
13505	13 x 75 mm	K2 EDTA K2 EDTA	2 ml	Lavender Viola		18 months mesi	100 / 1000
13520	13 x 75 mm	K2 EDTA K2 EDTA	2 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
135209	13 x 75 mm	K2 EDTA K2 EDTA	2 ml	Black Nero		18 months mesi	100 / 1000
13510	13 x 75 mm	K2 EDTA K2 EDTA	3 ml	Lavender Viola		18 months mesi	100 / 1000
135100	13 x 75 mm	K2 EDTA K2 EDTA	3 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
135201	13 x 75 mm	K2 EDTA K2 EDTA	3 ml	Black Nero		18 months mesi	100 / 1000
135300	13 x 75 mm	K2 EDTA K2 EDTA	4 ml	Lavender Viola		18 months mesi	100 / 1000
135306	13 x 75 mm	K2 EDTA K2 EDTA	4 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
135171	13 x 75 mm	K2 EDTA K2 EDTA	4 ml	Pink Rosa		18 months mesi	100 / 1000
135400	13 x 100 mm	K2 EDTA K2 EDTA	6 ml	Lavender Viola		18 months mesi	100 / 1000
135420	13 x 100 mm crossmatch	K2 EDTA K2 EDTA	6 ml	Fuchsia Fucsia		18 months mesi	100 / 1000
135560	13 x 100 mm	K2 EDTA K2 EDTA	6 ml	Light green Verde acqua		18 months mesi	100 / 1000
13560	16 x 100 mm	K2 EDTA K2 EDTA	9 ml	Lavender Viola		18 months mesi	100 / 1000



K3 EDTA TUBES PROVETTE CON K3 EDTA

CODE codice	SIZE dimensioni	DESCRIPTION descrizione	DRAWING aspirazione	COLOUR colore		SHELF-LIFE scadenza	PACKAGING confezion.
13001	13 x 75 mm	K3 EDTA K3 EDTA	1 ml	Lavender Viola		18 months mesi	100 / 1000
13005	13 x 75 mm	K3 EDTA K3 EDTA	2 ml	Lavender Viola		18 months mesi	100 / 1000
13840	13 x 75 mm IN	K3 EDTA K3 EDTA	2 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
130200	13 x 75 mm	K3 EDTA K3 EDTA	2 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
13148	13 x 75 mm	K3 EDTA K3 EDTA	2 ml	Black Nero		18 months mesi	100 / 1000
13010	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Lavender Viola		18 months mesi	100 / 1000
13125	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Translucent Lavender Viola trasparente		18 months mesi	100 / 1000
13150	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Black Nero		18 months mesi	100 / 1000
13161	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	White Bianco		18 months mesi	100 / 1000
131100	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Yellow Giallo		18 months mesi	100 / 1000
13025	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Dark green Verde scuro		18 months mesi	100 / 1000
13136	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Orange Arancio		18 months mesi	100 / 1000
13131	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Fuchsia Fucsia		18 months mesi	100 / 1000
13182	13 x 75 mm	K3 EDTA K3 EDTA	3 ml	Rusty Terra di Siena		18 months mesi	100 / 1000
13030	13 x 75 mm	K3 EDTA K3 EDTA	4 ml	Lavender Viola		18 months mesi	100 / 1000
13220	13 x 75 mm	K3 EDTA K3 EDTA	4 ml	Dark blue Blu		18 months mesi	100 / 1000
13040	13 x 100 mm	K3 EDTA K3 EDTA	6 ml	Lavender Viola		18 months mesi	100 / 1000
13060	16 x 100 mm	K3 EDTA K3 EDTA	9 ml	Lavender Viola		18 months mesi	100 / 1000

Recommended use

Mixing indications: immediately after blood collection, gently invert the sample 6-8 times.

Centrifugation: do not centrifuge.

Sample preservation:

Storage temperatures	Up to 24 °C	at 2-4 °C
Maximum preservation time	6 hours	≤ 24 hours

Before processing the haematological test mix carefully

To obtain plasma: centrifuge at 1300 g for 10 minutes at 20-25°C.

Sample preservation: for plasma separated from blood cells see following table:

Storage temperatures	Up to 24 °C	at 2-4 °C	≤ -20 °C
Maximum preservation time	≤ 8 hours	> 8 hours ≤ 48 hours	> 48 hours

Raccomandazioni d'uso

Indicazioni per miscelare: subito dopo il prelievo agitare 6-8 volte il campione per inversione lenta.

Centrifugazione: non centrifugare.

Conservazione del campione:

Temperature di conservazione	Fino a 24 °C	a 2-4 °C
Tempo massimo di conservazione	6 ore	≤ 24 ore

Prima di effettuare test ematologici miscelare accuratamente

Per ottenere plasma: centrifugare a 1300 g per 10 minuti a 20-25°C.

Conservazione del campione: seguire le indicazioni della tabella sottostante per il plasma separato dalla parte corpuscolata:

Temperature di conservazione	Fino a 24 °C	a 2-4 °C	≤ -20 °C
Tempo massimo di conservazione	≤ 8 ore	> 8 ore ≤ 48 ore	> 48 ore